

# PREVENTIVE MAINTENANCE CHECKLIST SIX (6) MONTH

SERIAL NO. \_\_\_\_\_ DATE \_\_\_\_\_

MACHINE I.D. NO. \_\_\_\_\_ MACHINE HOURS \_\_\_\_\_

TECHNICIAN(S) \_\_\_\_\_ LABOR HOURS \_\_\_\_\_

See the Referenced Section of the 2008T Preventive Maintenance Procedures (part number 508033 Rev. K) for detailed instructions to perform the following:

| <u>Reference Section</u> | <u>Procedure</u>   | <u>Value</u>                            | <u>Completed By:</u>                   |
|--------------------------|--|---|--|
| 2.1                      | Clean Filters.....(Replace If Necessary)                                   | _____                                   | _____                                  |
| 2.2                      | Inspect Pre-UF Pump filter.....(Replace If Necessary)                      | _____                                   | _____                                  |
| 2.3                      | Inspect UF Pump Check Valves.....(Replace If Necessary)                    | _____                                   | _____                                  |
| 2.4                      | Check High Voltage AC Connections.....(<0.2 ohms)                          | _____                                   | _____                                  |
| 2.5                      | Calibrate UF Pump.....(Range: 24 ± 0.1cc at 24 UF Pump strokes)            | _____                                   | _____                                  |
| 2.6                      | Check Conductivity Calibration.....(Range: Δ ± 0.1mS/cm)                   | _____<br><small>Machine Display</small> | _____<br><small>External Meter</small> |
| 2.7                      | Check Temperature Calibration.....(Range: 37 ± 0.3°C)                      | _____                                   | _____                                  |
|                          |  | (Range: 39 ± 0.3°C)                     | _____                                  |
| 2.8                      | Check Alarm Operation and Pressure Holding Test (N/A for BlueStar Premium) | _____                                   | _____                                  |
| 2.9                      | Verify pH (N/A for BlueStar Premium).....(Range 6.9 to 7.6)                | _____                                   | _____                                  |
| 2.10                     | Check Power Failure Alarm (N/A for BlueStar Premium).....                  | _____                                   | _____                                  |
| 2.11                     | Final Checks.....  | _____                                   | _____                                  |

**NOTES:**

---

---

---

---

---

---

---

---

---

---